



## LASER LEG VEIN CLEARANCE

Many people live with visible (and embarrassing!) leg veins. These leg veins can be safely and effectively treated with advanced laser technology.

The Palomar Lux1064™ Laser Handpiece offers a comfortable and effective leg vein treatment. This non-invasive process causes unsightly vessels to fade from view in a few treatments or less.

The Lux1064 Nd:YAG Laser Handpiece is an attachment for the Palomar StarLux™ Laser and Pulsed Light System.

ASK YOUR TREATMENT PROVIDER ABOUT  
**PALOMAR LASER  
LEG VEIN  
TREATMENTS**



Palomar Medical Technologies, Inc.  
82 Cambridge Street  
Burlington, MA 01803  
[www.palomarmedical.com](http://www.palomarmedical.com)

© 2007 Palomar Medical Technologies, Inc. Palomar and StarLux are registered trademarks and Lux1064 is a trademark of Palomar Medical Technologies, Inc. ALL RIGHTS RESERVED.

1553-0038 B



## YOUR GUIDE TO LASER LEG VEIN TREATMENTS

The Palomar StarLux® System  
with Lux1064™ Handpiece



Effective, Skin-safe,  
and Comfortable

## SEE THE RESULTS FOR YOURSELF: BEFORE AND AFTER PHOTOS



Pre Treatment



Post Treatment

Khaili Khatri, MD



Pre Treatment




Post Treatment

Khaili Khatri, MD



Pre Treatment

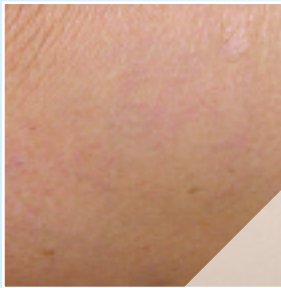

Post Treatment

David Vasily, MD



Pre Treatment



Post Treatment

Khaili Khatri, MD



## FREQUENTLY ASKED QUESTIONS

**How does Palomar laser leg vein treatment work?**  
Lasers deliver concentrated light in a specific wavelength of the spectrum. Each wavelength is absorbed by a different target, such as pigment, blood, or water. The Palomar Lux1064 handpiece delivers light pulses in the 1064 nm wavelength, which is ideal for targeting leg veins.

The light pulses heat the blood within the leg vein. The heat of the blood affects the vessel lining, which causes the vessel to constrict and eventually fade from view.

**What are Palomar laser leg vein treatments like?**  
Your treatment provider will move the Lux1064 handpiece over your skin, following the length of the vein, pulsing it all the while. Each pulse will send a beam of light through your skin. A flashing light, an audible tone, and a mild sensation on the skin will tell you when the handpiece is being pulsed.



**Are these treatments safe? Are they painful?**  
Your provider will go over the risks and benefits of treatment with you. Palomar leg vein treatments are skin-safe and comfortable. The Lux1064 delivers its energy in one smooth, gentle pulse, to protect the skin from a sharp rise in temperature.

In addition, a chilled fluid supply runs through the Lux1064 handpiece, down to its tip. Since the handpiece tip is in contact with the skin during treatment, your skin is continually cooled, so that you feel little heat from the laser pulses.

**What should I expect after treatment?**

- Immediately after treatment you will see any of these three possible outcomes:
  - The vessel will disappear
  - The vessel will darken
  - The vessel will shrink
- The surrounding skin may turn pink and be slightly swollen for up to 24 hours.
- Often more than one treatment is necessary to achieve full clearance of leg veins.
- Follow the instructions given by your treatment provider.

## OTHER STARLUX TREATMENTS

Your physician has chosen the Palomar StarLux Laser and Pulsed Light System because it offers the most versatile, effective, and comfortable aesthetic treatments available. In addition to leg vein clearance, StarLux offers such treatments as permanent hair reduction, photofacials for the clearance of vascular and pigmented lesions, acne, rosacea, and age spots.

